

Clean & Boost™

Dentin and Enamel Cleanser

Clean & Boost™ is an aqueous cleanser that has been designed to remove contaminants from the surface of a tooth prior to bonding. These contaminants include things such as: tooth debris, temporary cement, bacteria, blood, saliva, hand piece oil, imaging powder (used to create digital impressions for CEREC® restorations), etc.

Along with cleansing the surface, **Clean & Boost™** etches tooth surfaces to increase bond strengths of self-etch materials. Self-etch cements and many self-etch adhesives lack the ability to thoroughly prepare enamel. This lack of etching leads to staining of the margins, micro-leakage and accelerated marginal wear. **Clean & Boost™** properly prepares the enamel surface, resulting in improved long term results. While cleaning and etching, **Clean & Boost™** disinfects the surfaces being treated.

Clean & Boost™ is rinsed from the surface after use and thus is compatible with all other dental materials.

PRINCIPAL USES OF Clean & Boost™

- Cleansing dentin and enamel surfaces
- Removing imaging powder from all surfaces
- Disinfecting tooth surfaces
- Etching tooth structure for improved adhesion and long term results

Instructions for Use:

1. With the enclosed flow-thru brush tip, apply **Clean & Boost™** to the surface to be cleansed. Surface should be completely saturated with **Clean & Boost™**.
2. Using the delivery brush tip, agitate the **Clean & Boost™** on the surface for 10 seconds
3. Rinse thoroughly

For technical assistance please call toll-free: 1-877-APEX-123 (1-877-273-9123)