

## **Exquisite Restoration™**

### **Low Shrink, Light-cure Composite with Nanofiller Technology**

**Exquisite Restoration™** is a visible light-cure nanofilled composite for anterior and posterior restorations. Beside its low shrink and low abrasion **Exquisite Restoration™** has improved handling and features excellent mechanical properties. **Exquisite Restoration™** is highly radiopaque and is very wear resistant.

#### **Indications for Use:**

**Exquisite Restoration™** is suited for all Classes of direct restorations.

#### **Composite Fillings:**

1. Isolate and prepare the tooth surface to be restored. Insert a proper matrix where indicated.
2. Use a cavity cleanser such as **Clean & Boost™** to make the cavity as clean as possible.
3. Apply an adhesive, such as **Surpass® Self-Etch Adhesive system** or **Superb Universal Adhesive**, according to the manufacturer's instructions.
4. In Class I's and Class II's insert a thin layer (< 0.5 mm) of flowable composite (such as **Titan Flowable Composite**) over the entire pulpal floor up to and including the cavosurface margins. Activate the flowable composite according to the manufacturer's instructions.
5. Insert a 2 mm increment of the desired shade of **Exquisite Restoration™** and properly adapt to the cavity.
6. Light-cure for 20 seconds.
7. Continue inserting increments and light curing until cavity is slightly overfilled.
8. Finish and polish as usual.

#### **Note:**

Avoid the use of eugenol containing materials as eugenol can impair the hardening of resin materials and result in discoloration. Do not store resin based restoratives in the proximity of eugenol containing materials.

For technical assistance please call toll-free: 1-877-APEX-123